

# **PRO-AIR**

## ***Medical Supply & Equipment***

Phone: (989) 355-1601 / 1-844-445-1122

Fax: (989) 355-1605 / 1-844-284-5628

### **Nebulizer Instructions**

#### **Instructions for use:**

1. Assemble the nebulizer cup as instructed by the respiratory therapist
2. Connect the tubing and neb kit assembly to the compressor
3. Place the medication in the nebulizer cup
4. Sit upright
5. Turn on the compressor
6. Place the mouthpiece in your mouth
7. Breathe normally thru your mouth
8. After treatment, turn off the machine

#### **Cleaning Instructions:**

Rinse neb cup and mouthpiece after each treatment and allow them to air dry on a paper towel

#### **After the last treatment of the day:**

1. Completely disassemble the nebulizer cup
2. Wash the neb cup and mouthpiece in warm, soapy water
3. Rinse in warm water
4. Soak nebulizer cup in a solution of three (3) parts water to one (1) part white vinegar for thirty (30) minutes
5. Rinse in warm water
6. Shake excess water off the parts and lay them on a paper towel to air dry

#### **Precautions to help prevent infection:**

1. Do not leave medication in the nebulizer between treatments

2. Change nebulizer cup as often as your insurance allows: one non-disposable neb cup every six (6) months
3. Inspect the filter once a month – clean the filter with soap and water or replace it with a spare filter
4. If you need supplies or have any questions or problems, please call Pro-Air Medical Supply & Equipment at (989) 355-1601 or 1-844-445-1122